CVN Arctic Pallas Hypothermia Bag

Intended use:

Emergency extraction and prevention of hypothermia.

Intended users:

Military, EMS, Hospital, Refugee camps, Search & Rescue Teams, Police, Fire Service, Extreme sports

ENG: Pallas hypothermia bag instructions for use:

1. Unpack Pallas and straighten it, waxed bottom down.
2. Lift the edges of the Pallas and place the patient inside the bag. Closing edges on top of each other with velcro fastening. Leave the patient face visible.
3. Both sides of the bag have lifting loops for moving the patient.
4. The sides of the Pallas bag have treatment pockets (velcro fastening) and a plastic pocket for patient handling papers.
5. On the end of the patient's head, there is a fastening loop that allows the patient to be pulled using a harness.

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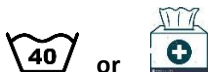
Take out the inner foam. Machine wash at maximum 40°C. Minor soiling use antibacterial wipes.

Store in a cool dry place, out of direct sunlight, off the floor.

Following each use thoroughly check for any compromise of fabric, particular attention to the stitching on the webbing, any compromise – DO NOT USE!

Adverse event notice:

Carevision Oy
Hiltusentie 26 A 2
90620 Oulu
FINAND
+358456431439
info@cvn.fi

Orvec International Limited
Malmo Road
Hull, HU7 0YF

CS Lifesciences Europe Ltd, 3 Inn Quay,
Dublin 7, Ireland, eurep@cslifesciences.com

Orvec International Limited
Malmo Road
Hull, HU7 0YF

CAREVISION OY NCAGE: A843G, NSN: 6530-58-001-437

3004493629

LATEX-FREE

MEDICAL SOLUTIONS
FOR DEMANDING SITUATIONS