

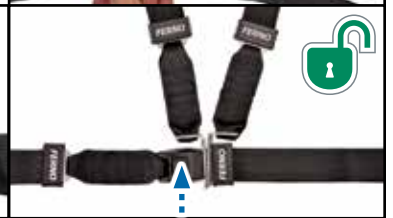
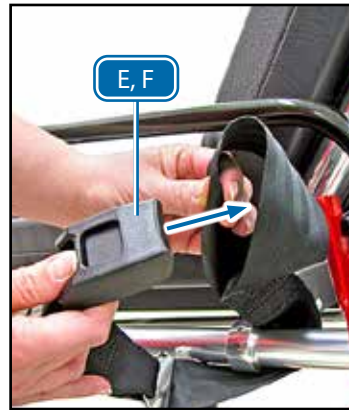
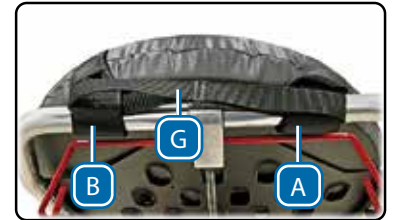
**Model 417-1** Harness Restraint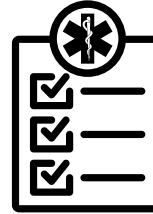

Ferno-Washington, Inc.  
70 Weil Way  
Wilmington, OH 45177-9371, U.S.A.  
1.937.382.1451

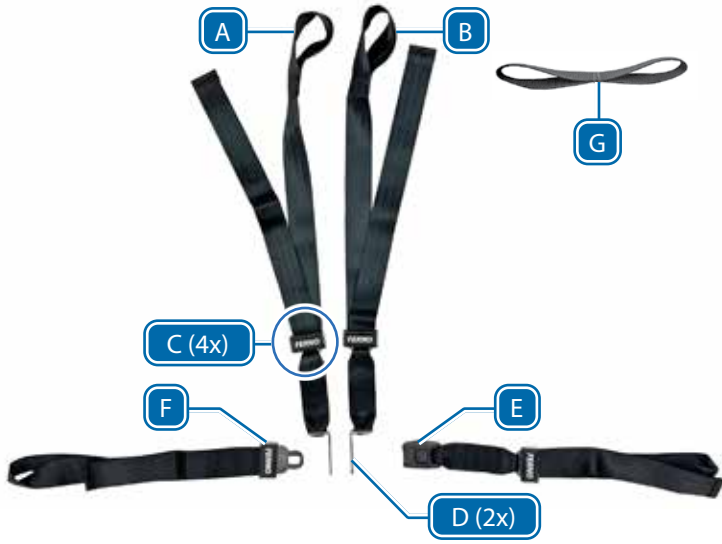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Ferno (UK) Ltd.  
Ferno House, Stubs Beck Lane, Cleckheaton,  
West Yorkshire, BD19 4TZ, UK  
+44 (0) 1274 851 999

## Description

Ferno® Model 417-1 harness restraint is a combination shoulder and chest restraint designed to help secure a patient on compatible equipment during transport in a motor vehicle. Use the harness restraint with additional restraints (not supplied) as part of a transport restraint system.

## General Guidelines for Use

Medical advice is beyond the parameters of this manual. It is the operators' responsibility to ensure safe practices during use of the product. Operators must:

- Read and understand this manual.
- Train and practice on proper use of the product following standard emergency patient-handling procedures. Keep training records.
- Inspect the product and confirm it is in good condition.
- Use the provided guide strap to help keep the shoulder straps from sliding off the patient's shoulders. Thread shoulder straps through the guide strap and position the guide strap near the backrest frame, not over the patient.
- For placement on a patient, follow local protocols. Adjust restraints to safely secure patient without causing discomfort or impairing circulation. Stay with patient at all times.
- Keep restraints fastened when not in use to prevent interference with equipment.

## Attaching the Restraint

- Attach chest straps (E, F) to cot main frame. Insert buckle (E) or receiver (F) through the loop and pull tight.
- Attach shoulder straps (A, B) to the cot backrest so the links (D) touch when assembled. Insert link (D) through the loop and pull tight. On an ambulance cot, attach guide strap (G) to both shoulder straps (A, B) to hold straps in place. Do not place the guide strap over the patient.
- Adjust length of each strap using slider (C).
- To buckle, insert tang (F) through links (D) into receiver (E). To unbuckle, Press release button and separate pieces.

## Load Capacity

 Refer to the equipment users' manual for the equipment load limit. Do not exceed equipment load capacity.

## Disinfecting and Cleaning

Remove restraints from the product before cleaning and disinfecting.

 Wipe all surfaces with disinfectant. Follow the disinfectant manufacturer's instructions for application method and contact time.

 Do not immerse metal slide bars or buckles in liquid. Immerse webbing in solution of warm water and a mild detergent. Rinse with warm, clear water. Dry with a towel or allow it to air-dry.

## Ferno Customer Relations

For ordering assistance or general information, contact your Ferno distributor or Ferno Customer Relations.

Information [www.ferno.com](http://www.ferno.com)  
 Authorized Service and Support [ferno.com/service-and-support](http://ferno.com/service-and-support)

U.S., Canada 1.877.733.0911  
 International +1.937.382.1451  
 Fax 1.888.388.1349

Complete English Users' Manual available at: [www.ferno.com](http://www.ferno.com).

**PATENTS**  
[ferno.com/patents](http://ferno.com/patents)

## WARNING

**Professional use by trained operators only. Use the product only for its intended purpose.**

**Improper use, operation, or maintenance can cause injury. Use, operate, and maintain only as instructed.**

**An unattended or unrestrained patient can be injured. Always stay with the patient and use patient restraints.**

Symbol	Definition	Symbol	Definition
	Follow training protocol		Lock
	Disinfecting		Unlock
	Cleaning		Lengthen
	Load capacity		Shorten

## NOTICE

Do not use disinfectants or cleaners containing phenolics or iodines. If bleach is used, rinse with plenty of clear water to remove bleach residue.

Do not use abrasive materials to clean the product.

Do not immerse metal buckles in liquids or place in washing-machine.

## Inspection Checklist

- Are all components present?
- Are buckles and webbing free of visible damage and operate properly?
- Can strap length be adjusted smoothly and properly?
- If used on a cot, is the guide strap properly attached and located so it will not interfere with equipment function?

If inspection shows damage or excessive wear, remove the product from service and replace.

Part #	Description
031-3914	417-1 Restraint, Orange
031-3915	417-1 Restraint, Black
031-3945	Guide Strap, Black (only)

## Adverse Event Notice

In the event of an adverse event or serious incident related to the use of this device, the end user/operator must report the incident to Ferno at 1-877-733-0911 or via email at [tscordinator@ferno.com](mailto:tscordinator@ferno.com). If the incident occurred outside of the U.S., report it to Ferno's Authorized Representative for that area.

## Disposal

Prior to disposal, clean and disinfect the product according to instructions in this manual. Recycle or dispose of the product following local protocols and regulations. Do not dispose of the product in municipal waste.

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