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E-mail: info@germa.se

www.germa.se

English: These directions for use may be updated without further notice.
Copies of the current version are available from the manufacturer.

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English Directions for use

1. Introduction

The Germa Navy Rescue Mattress is a patient handling equipment. The mattress can also be placed on top of a stretcher/trolley, used for patient transport.

The mattress design is in principally to protect patient prior to evacuation of patient and facilitate evacuation from the accident area.

The mattress is fitted with 7 handles on both sides plus 3 at head and footend. It is also equipped with four patient restraint straps, each with Fix-Lock, Quick-release plastic buckles. Additional Velcro closing inside the patient cover parts.

The mattress should be operated with a minimum of 2 trained operators when carrying and sliding the mattress a shorter distance, Additional people may be required for more challenging evacuations or heavier patients and when carrying longer distance. Additional carrying belts can be used to aid manual handling.

Safety rope and extra strap at head end for sliding and secure the mattress at all time at sea during patient handling.

In emergency situation the mattress can be moved or dragged in ergonomically advantageous on the ground using the carry handles or by connecting a carrying belt or rope to the mattress, making sure to avoid rough surfaces or sharp edges on the evacuation route of movement.

The mattress consists of an outer shell made from airtight material, which has a number of inner chambers, which contain granules. Granules are stored in pocket on each side as buoyancy aid.

The mattress design to facilitate evacuation and rescue in water. NOTE always minimum one rescuer to manage the mattress and patient in water.

Recommendation: Usable life of 3 years

2. Precautions

- Never mould the Mattress on the top of the head or under the foot soles as it might cause undesirable pressure against the body.
- On a regular basis check, all functions works correctly and physically test it by deploying the mattress by opening of cover part on unit, check all Velcro parts, safety belt and shell has no damages that could effect the capacity.
- Avoid handling the mattress next to sharp objects. If the mattress being pulled along the ground pay attention to obstacles on the ground, avoid if possible such as sharp glass, stones, sticks, glass, sharp edges etc.
- Check the Mattress and carrying straps for any visible damage or wear and tear. Repair any damage as required before re-using the mattress. Change any defect parts immediately. Ensure that all parts are present and operating including patient restraints and buckles, pillow and head strap
- Avoid handling the mattress next to open fire.
- On a current basis check that the mattress has no visible cracks.
- The mattress is unstable in water. You must never leave a patient unattended in water at any time.
- Max patient capacity 200 kg – lifting and carry
- Max maximized capacity as buoyancy aid 140 kg – NOTE: Always a rescuer by the patient in the mattress for safety.

3. Description

The mattress consists of an outer shell made of hard-wearing PVC coated fabric and a unique granulate-filled inner bag on each long side. The mattress is equipped with 7 handles on both sides and 3 handles at head and footend. It is also equipped with four colour coded patient restraint belts with Fix Lock buckles. Inside it is equipped with one cover part from foot end coming up over the patient plus side covers, all with Velcro closing. When not in use, store the belts and fasten them by the buckles.

When the mattress is not in use and not required to be used, the mattress can be folded and packed to be stored in a cabinet or mounted on the wall, easy accessible.

4. Operating the Mattress

1. Put the mattress on an even surface as possible close to the patient; avoid any sharp objects that may be on the ground or close to the patient area.
2. Position and adjust the mattress on the floor so that there is room for opening of all cover parts along all areas of the mattress, that it is flat ready to take the patient, with restraint straps unbuckled and not underneath the mattress
3. Place and position the injured patient in the medically desired position on the mattress, utilising standard adopted manual handling procedures, for safe transfer of the patient. Make sure to have safe protection of the patient head and body.
4. Check that the cover part are free and start to close and cover the patient from the footend cover, positioning up towards the head – cover over the chest up to the neck, close by cover by folding the side covers with appointed Velcro closing, make sure is tightened.
5. Secure the cover parts around the patient body, this to give effective protection, such that it becomes protective and comfortable for the patient. Ensure that the head strap stabilizes the head, and tighten comfortable.
6. Connect the patient restraint straps according to the colour code over the patient and cover parts, from legs to chest, and tighten them one by one.

Caution: Make sure that the mattress is not pressing too hard on patient body and bodily injuries.

7. Tighten the restraint straps further as the mattress becomes adapted to give better moulding protection around the patient. Check that there is no additional pressure on bodily injuries. Check that the unit is closed properly, give good protection and is comfortable for the patient prior to evacuation.
8. The patient is now ready to be evacuated and transported. NOTE always secure the mattress with the rope for safety – particularly transport over water or between units.

4.1 Use of the Mattress

1. Ensure that the mattress is secured purposefully prior to evacuation in staircases and over water.
 2. Preferably, place all rescuer in ergonomic position as possible when handling the patient in the mattress.
 3. When you place the mattress and patient on the stretcher; use the patient restraints on the stretcher to secure the patient.
- Max patient loading capacity 200 kg – lifting and carry
 - Maximized capacity as buoyancy aid 140 kg

5. Cleaning and Disinfecting

After each use the mattress should be wiped with a moist cloth and then dried. If dirty the mattress should be washed with soapy water and rinsed with clean water. Allow the mattress to dry completely before storage or usage. If the mattress needs disinfecting, first clean the mattress carefully and let it dry. Wipe the complete mattress with a solution containing 70% alcohol or use most common disinfects solutions.

6. Maintenance and Repair

At frequent regular intervals, the mattress must be checked for any visible damage.

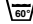
At the time, of checking the mattress must be tested that all functions are working according to Section 7.

Possible damage of side floating supports should be checked for by visual inspection, if any damage is found on the mattress, first clean the area around the damage and allow it to dry. Any damage on the side floating support must be repaired use of repair kit.

Find a repair patch of a suitable size from the repair kit and then spread the supplied glue all over the patch and all around the area of the mattress where the patch will be applied.

Wait for the glue to become "dust dry" (approx. 2-3 minutes) and then place the patch on the area of glue and leakage. Apply pressure to the whole area of the patch; making sure good contact has been established in the whole area and that any air bubbles are removed.

The repair will have full strength after 24 hours, but can be used within a few minutes in case of emergency

Note: Patient restraint straps are detachable, washable  and replaceable, washing in laundry bag, see label at the foot end restraint. If blood gets on the straps, soak in cold water first before washing

7. Testing the Mattress

Straighten out the mattress on the floor. Any discovered damage on the mattress, cover part, Velcro parts or safety belts rigidity repaired or exchanged. Indication of damage in the mattress from the side floating support to be repaired before use. Check both for damages and correct functions.

See repair kit

8. Storage

After each use, check to ensure that the mattress is still operating correctly and made ready for its next use. Make sure the granules are distributed evenly in the side support, and then fold the mattress in its initial position.

The mattress can be stored on top of the stretcher, on the wall or folded in a storage bag, such that it will fit in the designated storage area.

9. Specifications

Dimensions:

Length x Width x Height: see product information below

Weight: see product information below

Heat resistance: 70°C

Cold resistance: -30°C

Materials:

Part

Top and bottom material: Polyester fibres PVC coated (flame resistant)

Extra reinforcement material: Polyester fibres PVC coated (flame resistant)

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Report no. :

(To be assigned in by
Manufacturer)

Please print or type or fill in by MS Word, all sections a. – i. Fax or e-mail to your supplier. Please use repair order when reporting repairs not associated with complaints.

For claims please specify :
Purchase Date

Description of product

Description of Incident/Problem: 1) What, 2) How?

When? In connection with: 1) transport, 2) unpacking, 3) preparing, 4) test, 5) cleaning, 6) normal use, 7) training, 8) disassembly, 9) assembly, 10) storage, 11) other:

Info on the product, prior to problem,

Storage: 1) stock, 2) site of use, 3) packed, 4) unpacked, 5) softpack, 6) other:

Used cleaning and disinfecting agents/Sterilizing method:

Pattern of use: 1) stand-by for emergency, 2) routine use, 3) stand by, 4) other:

Environment: 1) Indoor, 2) Outdoor, 3) Heat, 4) Cold, 5) Humidity, 6) Sunlight, 7) Gases, 8) other:

Which **actions** have been: a) taken, b) planned by the reporter or user at the site of incident?

1) examination of the product on site, 2) replacement of the product, 3) other:

Suggested actions by the distributor/complainant

Returning of samples/faulty product(s),

- » 1) Already shipped/Will be shipped to _____,
- » 2) Pending request from manufacturer,
- » 3) Product/parts not available:

Date:

We acknowledge the receipt of this report by manufacturer and will start processing it immediately. Please use the assigned complaint report number for further reference.

Your contact person in this matter is: _____

Name:

Direct Tel.

Direct Fax.

Init/Date: