DIRECTIONS FOR USE

Heavy Duty Carry / Slide sheet

English: Directions for use, **Ferno Carry / Slide Sheet Heavy Duty, Large**
Svenska: Användar Manual, **Ferno Carry / Slide Sheet Heavy Duty, Large**
Dansk: Brugsanvisning, **Ferno Carry / Slide Sheet Heavy Duty, Large**
Suomi: Käyttöohje, **Ferno Carry / Slide Sheet Heavy Duty, Large**
Norsk: Brukerveiledning, **Ferno Carry / Slide Sheet Heavy Duty, Large**

Distributor:

Ferno Norden AS
Innlaget 298
NO-3185 Skoppum
Norway

Phone: +47 33 03 45 00

Fax: +47 33 03 45 10

E-mail: fno@fernonorden.com

www.fernonorden.com

Manufacturer:

AB Germa
Industrigatan 54-56
SE-291 36 Kristianstad
Sweden

Phone: +46 (0)44 12 30 30

Fax: +46 (0)44 10 31 79

E-mail: info@germa.se

www.germa.se

124429006666, 2019-11-28, version 3

English Directions for use

1. Warning and caution statements

CAUTION

- Avoid handling the mattress next to sharp objects.
- Avoid handling the mattress next to open fire.
- On a current basis check that the Carry Slide sheet has no cracks or damages.
- Own modifications on products not allowed – Your medical specialist to be responsible for any modification.
- When decided to take the product out of service and destroy – make sure to follow your local instructions or contact us for additional information.

Recommendation: Usable life of 5 years

2. Intended use

The **Ferno Carry Slide Sheet, large** is intended to be used for heavy duty patient handling as slide/ lifting /carry sheet to or from ground or bed to a stretcher for transportation of a patient. It is also a device intended to facilitate the transfer of the patient from one stretcher to another.

The sheet can be carried by minimum two or more persons for a shorter distance, and a longer distance if two or more persons use additional carrying handles and belts. You can pull the sheet on the ground for shorter distances by the handles or by connecting a carrying belt or rope.

The mattress consists of a double coted PVC, fully welded, material that is fitted with handles on four sides for lifting, carrying, sliding or pulling including reinforced back.

Max loading capacity 380 kg

3. Specifications

The Germa Carry Slide sheet is tested in accordance with EN 1021-1

Temperature range: -30°C to +70°C

Parts/Materials

Outer shell: PVC coated fabric (flame retardant)

Rope inside handle nylon

Reinforced back of fiberglas

Dimensions	Length (cm)	Width (cm)	Height (cm)	Weight (kg)	N °of handles
Germa Carry Slide sheet 12 handles	205	143	0,3	4,3	20

4. Description

The Germa Carry Slide sheet is equipped with 6 and 4 handles at each side.

The mattress consists of an outer cover of hard-wearing PVC coated polyester weaved. Including reinforced back of the sheet.

5. Operating instructions

1. Place the injured patient in the medically desired position on the Carry Slide sheet.
2. Grasp the handles firmly slide, lift and carry the sheet and patient carefully on to the stretcher.
3. Close the harsness system of the stretcher accproding to stretcher instructions.
4. Check that the unit is stable and comfortable for the patient.
5. The injured patient is ready to be transported on the stretcher.

6. The Germa Carry Slide sheet can not be used as a transfer tool. Its dimensions allows it to fit on any commonly sold heavy duty stretcher with mattress.
7. The Germa Carry Slide sheet can be slided over to another stretcher or trolley at the hospital or at change of transportation tool.

6. Cleaning and Disinfecting

At a regular basis, and after use the sheet should be wiped with a moist cloth and dried, if dirty, the mattress may be washed with soapy water and rinsed with clean water.

Allow the mattress to dry before storage.

If the transfer mattress needs disinfection first clean the mattress carefully and let it dry. Pour a solution containing 70 % alcohol or other common approved disinfectants on a clean cloth. Smear the transfer mattress with the cloth.

7. Repair of Carry Slide sheet

If small surface damage occurs repair as follows:

- a) Find a patch in a suitable size and put glue on both the patch and the damaged spot on the transfer mattress.
- b) Wait for the glue to become dust dry (approx.2 min.) and place the patch on the damaged spot.
- c) Press firmly the whole area of the patch, so that there is a good contact and all air is pressed out under the patch.

The repair will receive full strength after 24 hours.

For all other damages the Carry Slide sheet shall be exchanged.

8. Storage

The Germa Carry Slide sheet can be positioned on top of the stretcher or stored in the ambulance with a cover.

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Report no. :

(To be assigned in by
Manufacturer)

Please print or type or fill in by MS Word, all sections a. – i. Fax or e-mail to your supplier. Please use repair order when reporting repairs not associated with complaints.

For claims please specify :

Purchase Date

Description of product

Description of Incident/Problem: 1) What, 2) How?

When? In connection with: 1) transport, 2) unpacking, 3) preparing, 4) test, 5) cleaning, 6) normal use, 7) training, 8) disassembly, 9) assembly, 10) storage, 11) other:

Info on the product, prior to problem,

Storage: 1) stock, 2) site of use, 3) packed, 4) unpacked, 5) softpack, 6) other:

Used cleaning and disinfecting agents/Sterilizing method:

Pattern of use: 1) stand-by for emergency, 2) routine use, 3) stand by, 4) other:

Environment: 1) Indoor, 2) Outdoor, 3) Heat, 4) Cold, 5) Humidity, 6) Sunlight, 7) Gases, 8) other:

Which **actions** have been: a) taken, b) planned by the reporter or user at the site of incident?

1) examination of the product on site, 2) replacement of the product, 3) other:

Suggested actions by the distributor/complainant

Returning of samples/faulty product(s),

- 1) Already shipped/Will be shipped to _____,
- 2) Pending request from manufacturer,
- 3) Product/parts not available:

Date:

We acknowledge the receipt of this report by manufacturer and will start processing it immediately. Please use the assigned complaint report number for further reference.

Your contact person in this matter is: _____

Name:

Direct Tel.

Direct Fax.

Init/Date: