


STANAG 2040 4-FOLD NATO Stretcher MANUAL

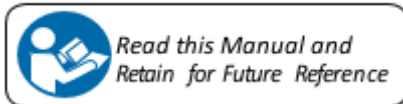



TIP: Regular cleaning helps reduce the risk of transmitting disease and enables the equipment to function at its optimum

Disclaimer

This manual contains general instructions for the use, operation and care of this stretcher. Safe and proper use of this stretcher is solely at the discretion of the user. Safety information is included as a service to the user. All other safety measures taken by the user should be within and under consideration of applicable regulations. It is recommended that training on the proper use of this stretcher be provided before using this stretcher in an actual situation. Retain this manual for future reference. Include it with the stretcher in the event of transfer to new users. Additional free copies are available upon request from FERNO Manufacturing.

Model STANAG 2040 4-FOLD NATO Stretcher Product Specifications

Length with extended handles	2293 mm
Length with retracted handles	2010 mm
Length of mesh sheet	1910 mm
Width of stretcher	580 mm
Height - open	145 mm
Folded length	190 mm
Folded width	230 mm
Folded height	530 mm
Weight	9,3 kg
Maximum load capacity	250 kg

WARNING

Untrained operators can cause injury or be injured. Permit only trained personnel to operate the stretcher.

Improper use of the stretcher can cause injury. Use the stretcher only for the purpose described in this manual.

Improper operation can cause injury. Operate the stretcher only as described in this manual.

An unattended patient can be injured. Stay with the patient at all times.

An unrestrained patient can fall off the stretcher and be injured. Use restraints to secure the patient on the stretcher.

An improperly-secured stretcher can separate from the transporter and cause injury. Attach the stretcher securely and verify that the slam latches have locked.

Helpers can cause injury. Maintain control of the stretcher, operate the controls and direct all helpers.

Helpers can be injured. Show helpers where to grasp the stretcher to avoid pinch points.

Improper maintenance can cause injury. Maintain the stretcher only as described in this manual.

Improper parts and service can cause injury. Use only Ferno parts and Ferno- approved service on the stretcher.

Modifying the stretcher can cause injury and damage. Use the stretcher only as designed by Ferno.



STANAG 2040 4-FOLD NATO Stretcher Instructions

[Product Name] Medical Device STANAG 2040 4-FOLD NATO Stretcher

[Intended Use]

The Model STANAG 2040 4-FOLD NATO Stretcher is an emergency patient-handling device. Each stretcher in the series is for professional use by a minimum of two trained operators.

In this manual, the generic term “stretcher” is used when the feature is shared by all models in the series. The individual stretcher name is used when features differ.

[Application]

The Model STANAG 2040 NATO 4-FOLD Stretcher is the emergency stretcher of choice for: Military Medics, Athletic Trainers, First Responders, EMS, Wilderness Rescue, Ski Patrols, Outdoor Adventurers, Safety Engineers, Veterinarians and more.



[Directions for use]

Extending the STANAG 2040 4-FOLD NATO STRETCHER



Fully collapsed.



Ensure that the handles are pointing upward.



Extend the stretcher by pulling the handles outward.



Secure the locking arm by pushing the center hinge into position. Now, flip the stretcher onto its feet.

Collapsing the STANAG 2040 4-FOLD NATO STRETCHER

When collapsing ensure all straps are locked and the fabric remains taut as to avoid bunching over the hinges (which restricts folding).



Fabric incorrectly hangs over hinge.



Unlock hinge by pushing toward end. Twist handles inward.



Raise center hinge.





Fold center onto the end and then fold over other end.



Draw all ends together to form a tight bundle.
Use the restraints to lock the bundle closed.

EXTEND/RETRACT THE HANDLES



To extend the handle, twist the end and then pull out.



To retract the handle, push in the pin at the base and slide the handle inward.

Care must be taken during assembly and disassembly to avoid injuries.



[Cleaning and Disinfection]

Wipe all surfaces with disinfectant. Follow the disinfectant manufacturer's instructions for application method and contact time. Ferno recommends you inspect the stretcher for damage as you disinfect it.

[Inspecting the stretcher]

Have your service's equipment maintenance personnel inspect the stretcher regularly.

If inspection shows damage or excessive wear, remove the stretcher from service until repair is made.

[Parts and services]

To order Ferno parts and for professional stretcher repair, contact FERNO or your Ferno distributor. Your distributor is the only agent authorized by Ferno to manage, service and repair Ferno products.

Ferno Slovakia Contact:

Ferno Slovakia s.r.o. Bošáca 893
913 07 Slovakia
Tel: + 421 (0) 32 7708 010

[Declaration of Conformity]

Ferno Slovakia s.r.o. declares that the product conforms to STANAG 2040 and meets requirements for roadworthiness (10G-tested).



- STANAG 2040 4-FOLD NATO STRETCHER (BLACK) w/BIO 4-point restraints / 60-0122-001
- STANAG 2040 4-FOLD NATO STRETCHER (BLACK) w/BIO 2 restraints / 60-0122-002
(does not meet STANAG 2040 Chapter 2 road-worthiness)
- STANAG 2040 4-FOLD NATO STRETCHER (GREEN) w/BIO 4-point restraints / 60-0122-004
- STANAG 2040 4-FOLD NATO STRETCHER (BLACK) w/4-point restraints / 60-0122-006

Accessories:

- WHEELED STRETCHER UNDERCARRIAGE ARION / 65-0440-001
- BACKPACK FOR STANAG 2040 4-FOLD STRETCHER / 65-0317-004
- STANAG 4-FOLD STRETCHER IV POLE / 65-1500-050

