INSTRUCTION FOR USE

ENG

THERMOSAVER

Thermosaver TSR 5

Hypothermic Rescue Bag

Please read all instructions before use.

These instructions for use describe use of the Thermosaver Hypothermic Rescue Bag. Users must read these instructions for use carefully prior to using the Thermosaver for the first time so that all features are thoroughly understood. Please keep these instructions for use for future reference.

Indications for Use

Patient should be over 5 years of age.
Patient is suffering or likely to suffer from hypothermia.
Patient needs aid to prevent heat loss from their body.




Contraindications

Not to be used by children under 5 years.
Not to be used on multiple patients.

Classification

Thermosaver is classified as Class I according to the Medical Devices Regulation (MDR 2017/745).

Thermosaver only accepts responsibility for the equipment's safety, usability, and performance if:
- The equipment is used in accordance with its intended use, and
- The equipment is used in accordance with the product documentation

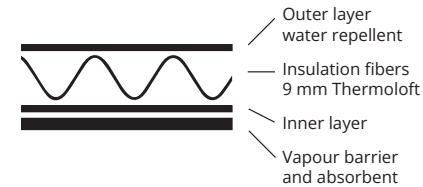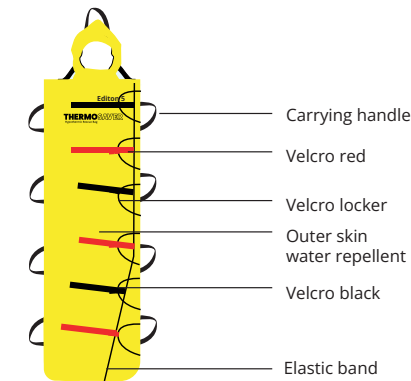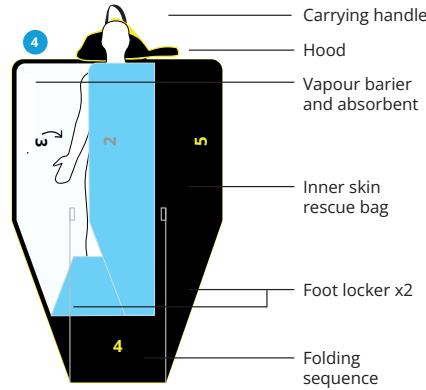Intended Use

The Thermosaver is intended to be used on patients suffering from or likely to suffer from hypothermia and stops heat loss. The device is for single use.

Intended User

The intended user is the trained first responder familiar with the content of Thermosaver's Instruction Manual. The device is not intended for lay-users.

Patients	Applies	Does not apply	Description
Neonatal		x	
Children	x		The device it not intended for persons under 5 years
Adults	x		
Disabled persons	x		
Operators	Applies	Does not apply	Description
Health Care Professionals*	x		
Certification required		x	The IFU is sufficient to ensure proper use.

*Operators are trained first responders (e.g. emergency services, police or fire brigade officers as well as volunteers with land or sea rescue units).

Operational Safety

The Thermosaver should only be operated by trained professionals with training in rescue operations. Such personnel is referred to as operator throughout this manual. Do not expose to open fire.

Patient Safety

The Thermosaver is for single use. The device should not be used on multiple patients.

Safety and adverse events

Operator must ensure that the instructions are followed appropriately to avoid an unlikely event of suffocation.

Safety regulations

General safety regulations.
Dispose of the used product in accordance with accepted medical practice and applicable local and national regulations. Used product may be contaminated and represent a potential biohazard.
The Thermosaver has a shelf life of 5 years.
Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State on which the user and/or patient is established.

Environmental and Handling Conditions

Operation temperature range	-40°C to +50 °C
Storage temperature range	-40 to +70 °C
Maximum relative humidity for storage	95%

Symbols used on labels

Symbol	Meaning	Symbol	Meaning
	Manufacturer name and address		Last use date
	Do not reuse		Unique device identifier (UDI Carrier)
	Serial Number		Product is a medical device
	Model number		Country of Origin
	Keep away from open fire		Link to IFU
	WARNING: use return bag		

Package and Device Inspection

The operator is required to:
- Ensure packaging is intact. Do not use if it appears to be damaged.
- Ensure that the Thermosaver is within its expiration date.
- Visually inspect the Thermosaver for any signs of damages to the bag itself. Do not use the Thermosaver if it appears to be damaged.

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